



Women's International Pharmacy

Custom Compounded Prescriptions for Men and Women

How to Use the TopiClick® Dispenser

To Open:

Pop cap off by placing thumb on lid and pushing upward, see Picture A (on reverse side).

To Dispense:

Turn the base in the direction shown in Picture B (on reverse side) until you feel and/or hear a click. Your prescription label will indicate how many clicks it takes to obtain the prescribed dose.

1 click = $\frac{1}{4}$ gram

2 clicks = $\frac{1}{2}$ gram

3 clicks = $\frac{3}{4}$ gram

4 clicks = 1 gram

NOTE: When you first begin a new TopiClick®, a few turns may be required to bring contents up to the dispensing opening, before you can start counting your clicks.

To Apply:

Once you have clicked out your dose, apply dose by rubbing applicator to the area of skin as directed by practitioner, see Picture C (on reverse side).


To Store:

Replace cap.

To Dispose:

TopiClicks® are not reusable.

 TopiClicks® are recyclable; Category 5.

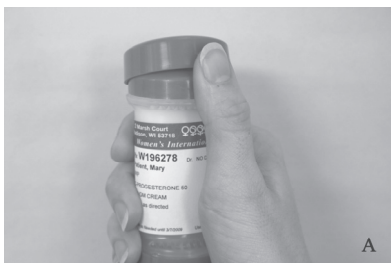

Women's International Pharmacy

www.womensinternational.com

3/08